

PLASHINE FILL Not Experienced
With Hyaluronic Acid Fillers
Elasticity and jawline contours
for up to two years.

PLA
SHINE
FILL

PLASHINE FILL
Purified PLLA with HA



Purified PLLA with HA Bio Collagen stimulator

Highly concentrated bio stimulator restores the skin from deep layer, preventing sagging tissues and improving skin elasticity. Unable to experience this with hyaluronic acid fillers that temporarily fill wrinkles, the PLASHINE FILL Series is a collagen stimulator that can improve skin radiance, elasticity, and jawline contour for up to two years

- Excellent Biocompatibility
- Safety and Effective (PLLA is completely approved by FDA)
- Excellent Soft Tissue Augmentation (Improve Wrinkle, Scar, Skin Pores)

- Various product dosages (face, body)
- Faster hydration speed compared to equivalent products (15~30min, 3000RPM)
- Low and uniform injection force (0.9 ~ 2.0N)



PLASHINE FILL Protocol

A. PLASHINE FILL 200mg + Normal saline 12cc (Base solution)

The original vial bottle is not enough to fill 12cc of N/S

And so, I recommend to use a 20cc of vacant vial bottle (like vacant 20cc lidocaine bottle).



1. Pore tightening injection

32~34G needle intradermal injection or multi needle injector (like skin booster)

Please contact us if you need multi-needle injecting machine




2. Scar improvement injection (for acne scar)

How to apply: 32~34G needle intradermal injection (to elevate the depressed part of scar)




3. Collagen stimulator

How to apply: 21~24G cannula infiltration on subcutaneous layer (like Gouri®, Profilo®)

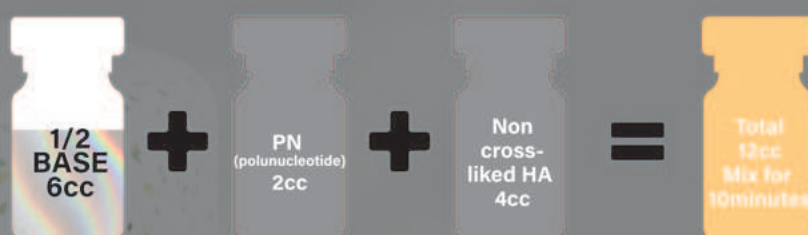

PLASHINE FILL Application

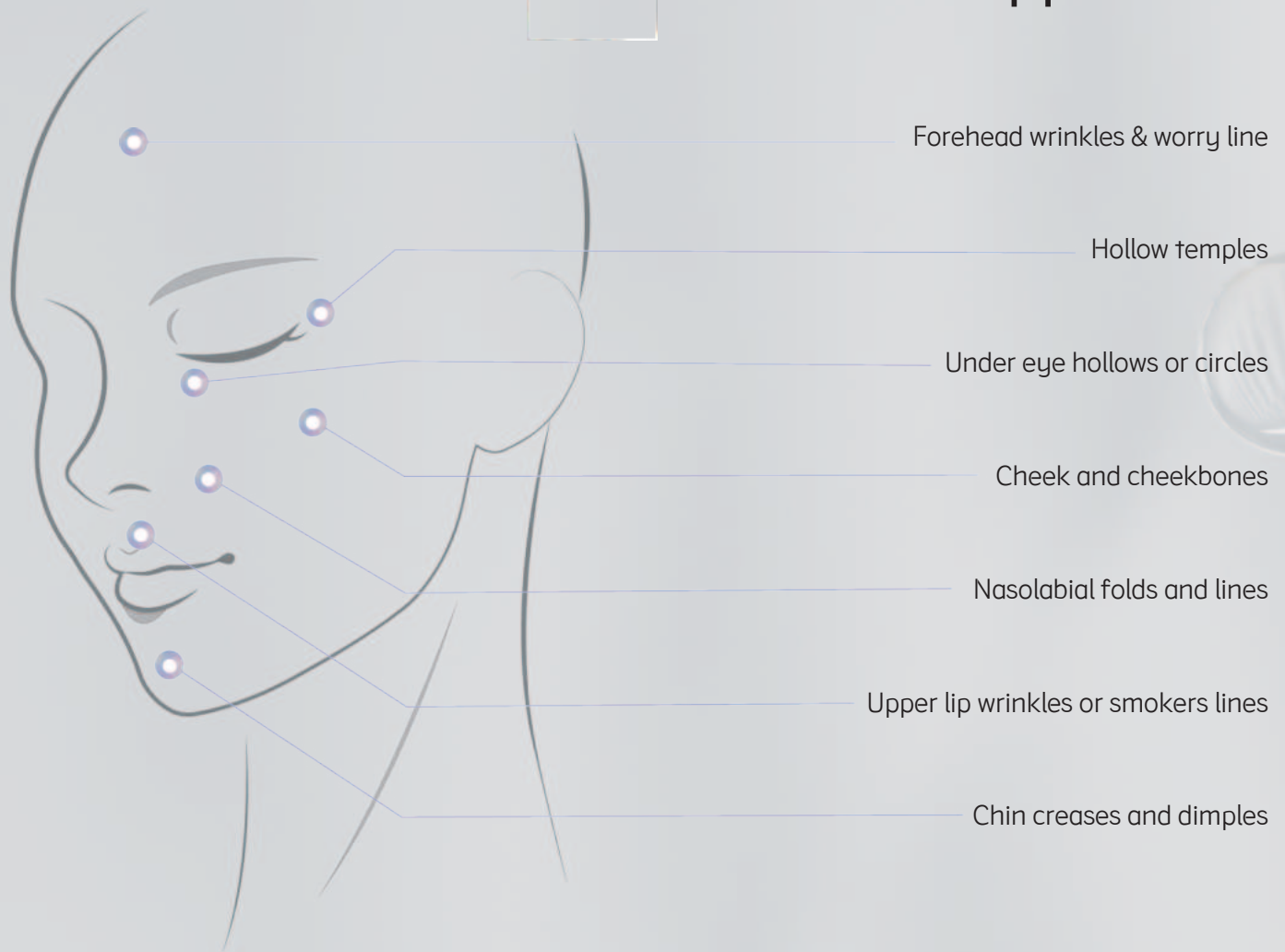

Contents	"PLASHINE FILL" PLLA Bio Filler
Particle size	~30um(microsphere)
Package	200mg /Vial
PLLA Contents	170mg
Infusion volume (0.9% NaCl Saline)	3ml
Hydration time	15~30min
pH	4.0~7.0
Osmotic pressure	250~350mOsm/kg
Normal force (N)	0.9~2.0N