

PURAVELLE


Cross Linked Hyaluronic Acid



Classic


Deep

Volume

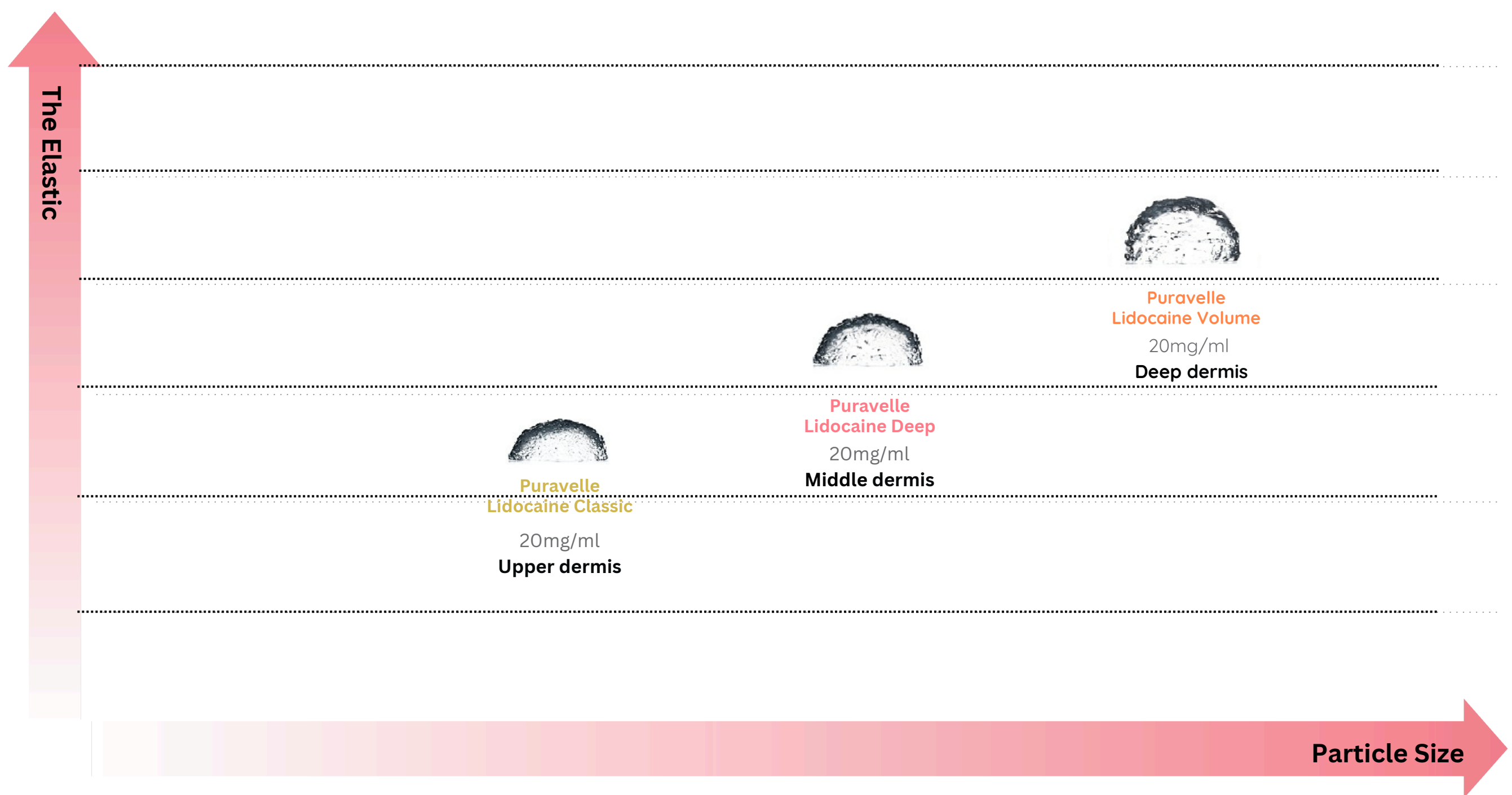

Cross Linked Hyaluronic Acid Dermal Filler



- Purevella Lidocaine contains non-animal-based hyaluronic acid with a molecular weight of 2,500,000Da as its active ingredient.
- PURAVELLE Lidocaine’s innovative technology enables the simultaneous achievement of optimal viscosity and elasticity by forming a 3D structure with a minimal amount of cross-linking agent.

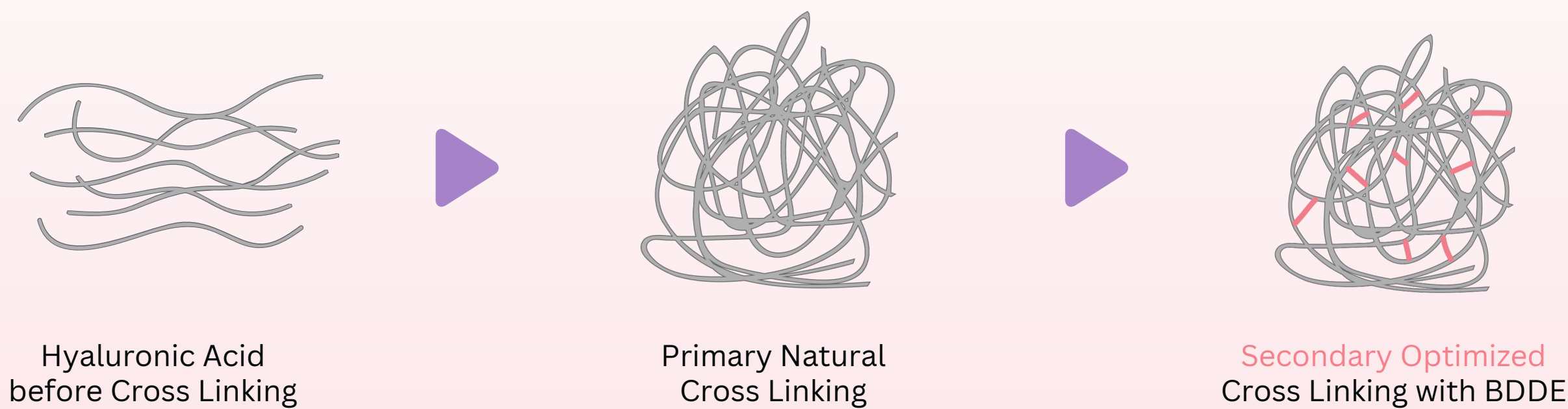

Particle Plastic Process

- PURAVELLE Lidocaine ensures a uniform particle structure with its proprietary HA cross-linking method.
- The PP process is designed to standardize HA particle formation. By applying this advanced technology, PURAVELLE Lidocaine maintains a precise and consistent HA particle size. Such uniformity improves injection stability, molding precision, and overall treatment performance.

Before & After

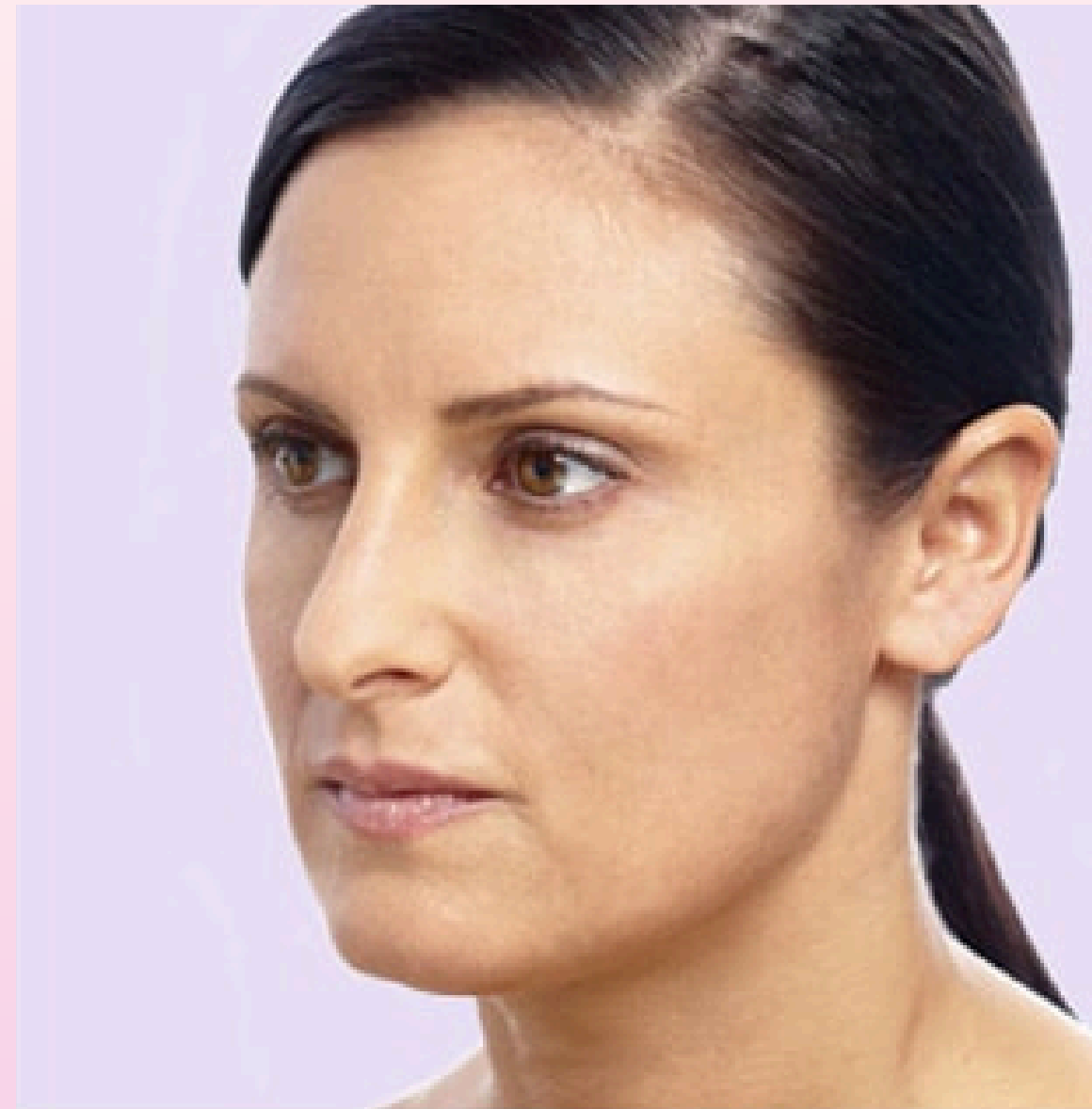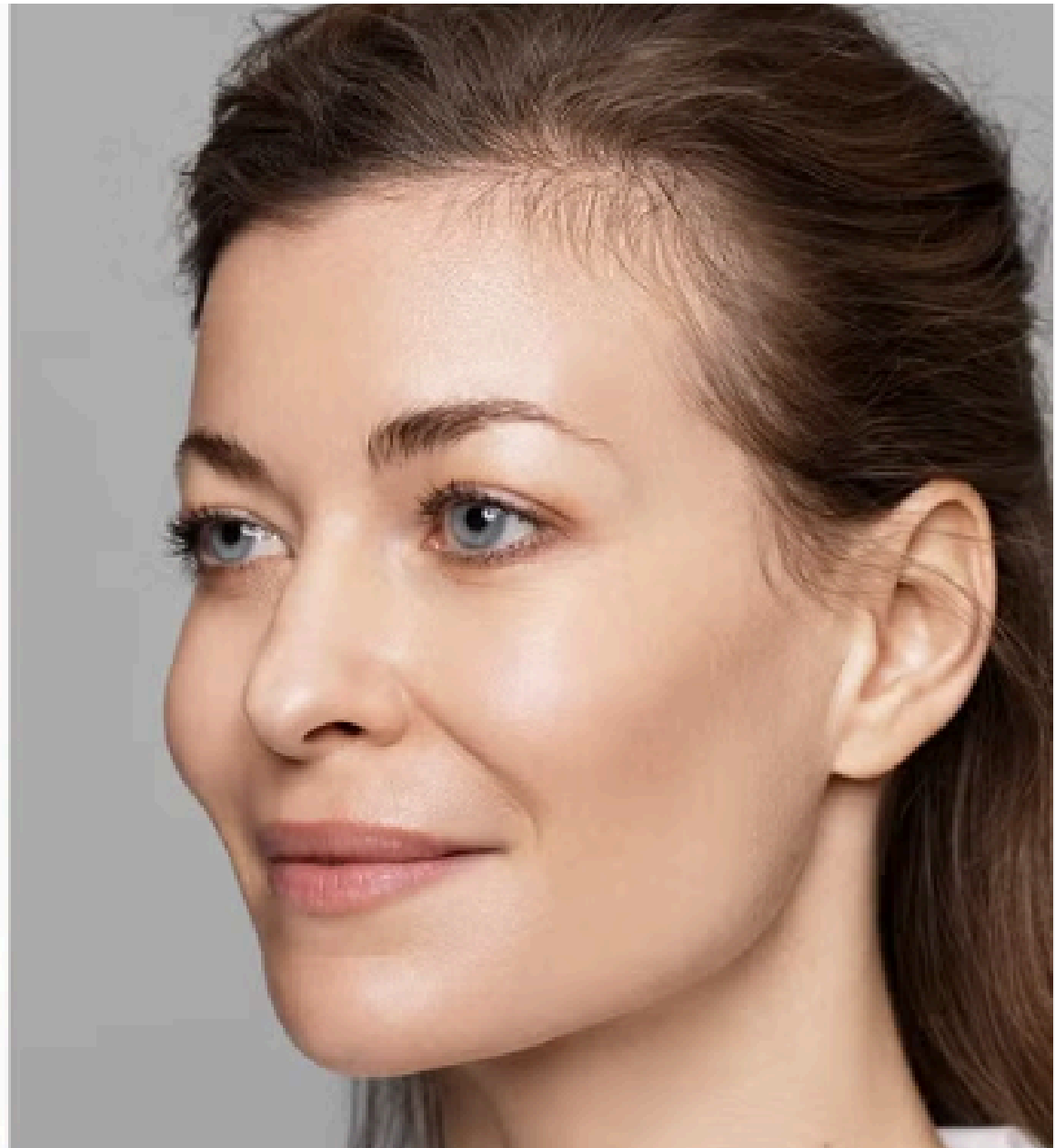

Injection Area




Example of procedure

Forehead - Deep/ Volume

Cannula: 23 or 25G cannula

Injection amount: 2cc (Min 1cc Max 4cc)

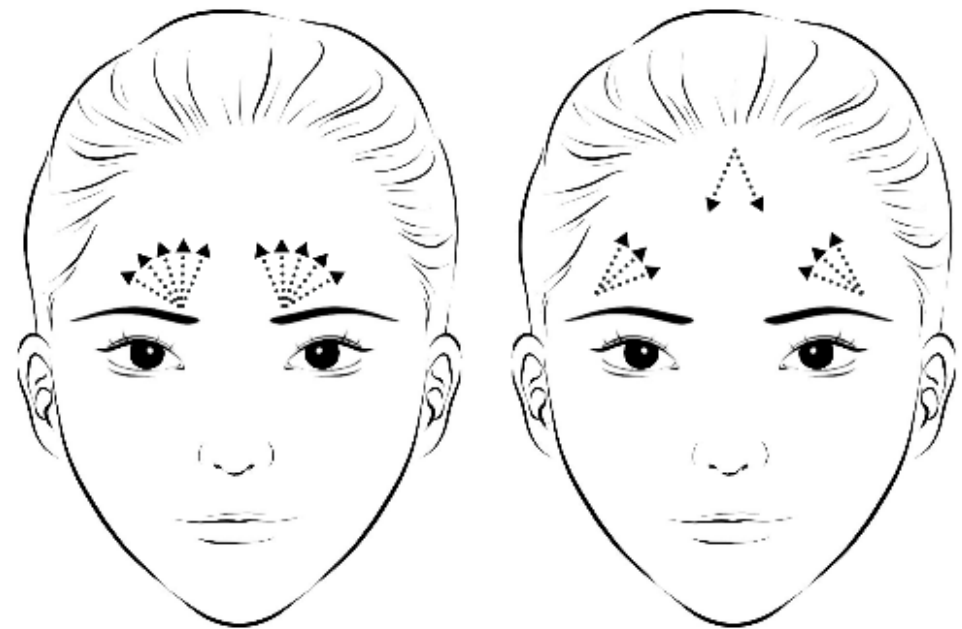


Temple - Deep/ Volume

Cannula: 23/ 25G cannula

Needle: 27G Needle

Injection amount: 2cc (Min 2cc Max 4cc)

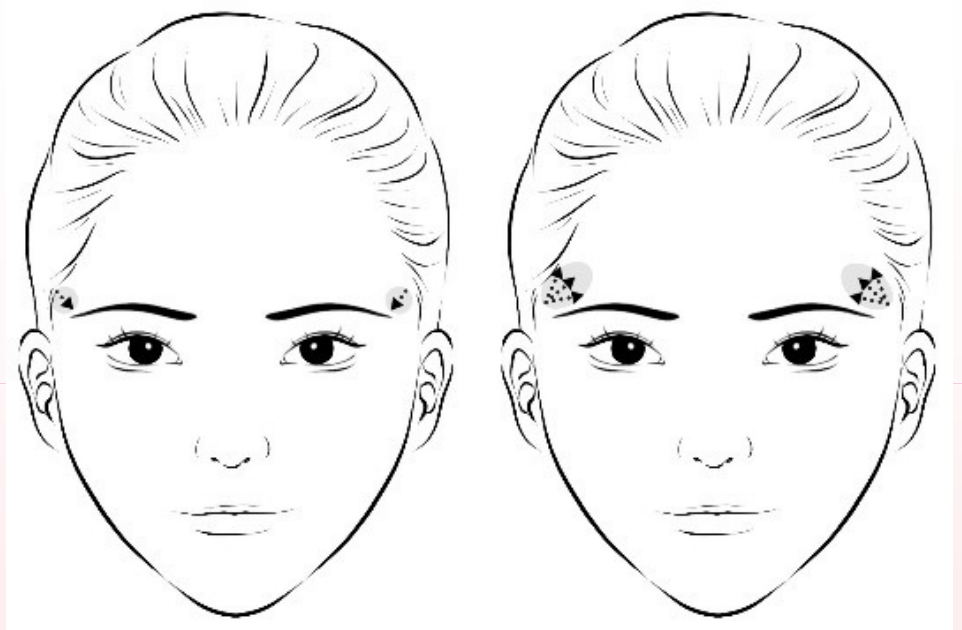


Tear Trough - Classic

Cannula: 25 or 27G cannula

Needle: 27 or 30G Needle

Injection amount: 0.6cc (Min 0.3cc- Max 1cc)

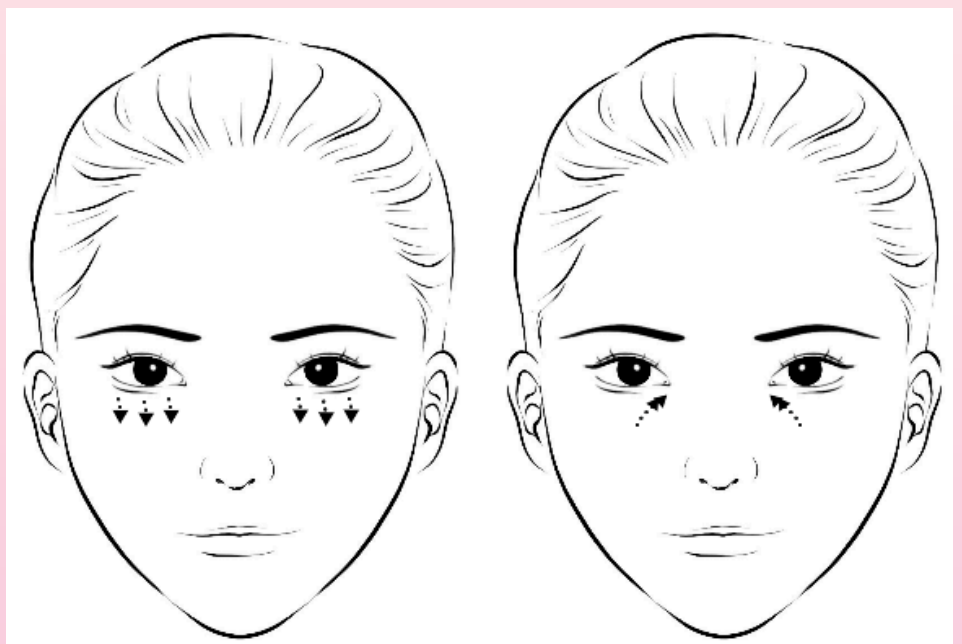

Example of procedure


Cheek - Deep/ Volume

Cannula: 23 or 25G cannula

Needle: 25 or 27G needle

Injection amount:

- Anteromedial cheek: 2cc (Min 0.6cc Max 3cc)
- Submalar region: 3cc (Min 1cc Max 4cc)

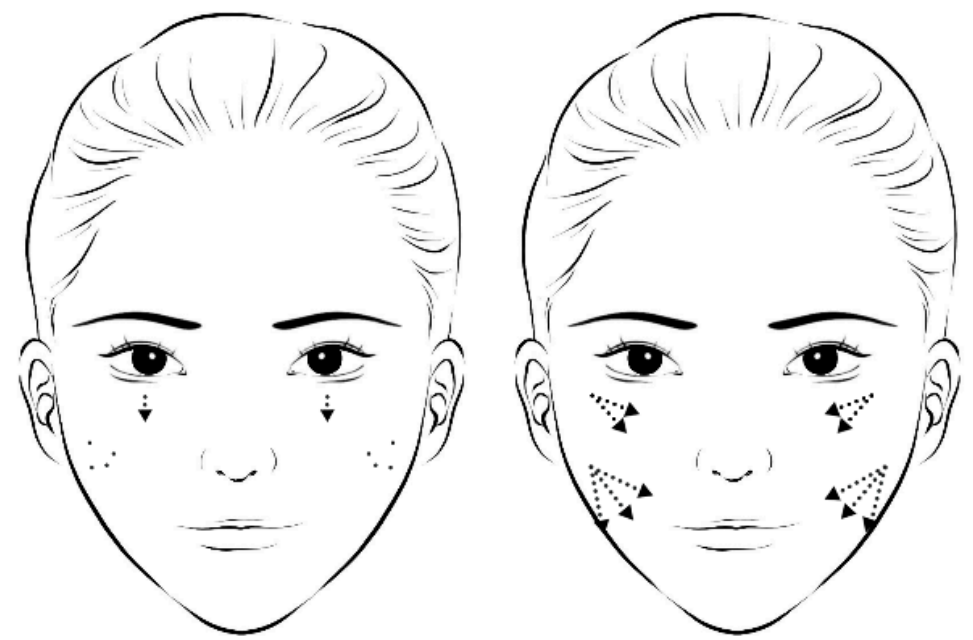


Nasolabial folds - Deep/ Volume

Cannula: 23 or 25G cannula

Needle: 25 or 27G needle

Injection amount: 2cc (Min 1cc Max 3cc)

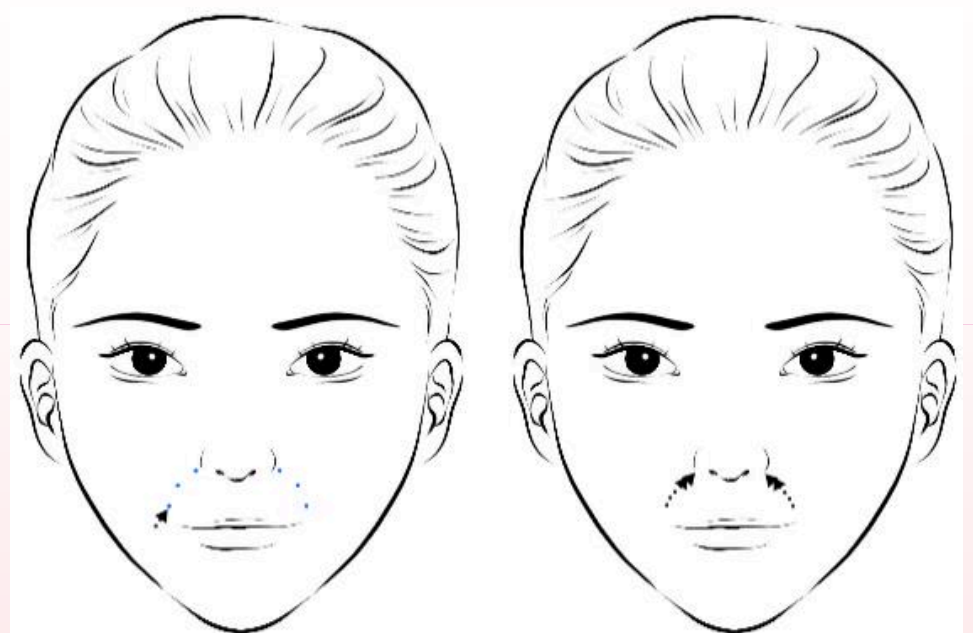


Lip - Fine/ Deep

Cannula: 25 or 27G cannula

Needle: 27 or 30G Needle

Injection amount: 0.6cc (Min 0.5cc- Max 1cc)

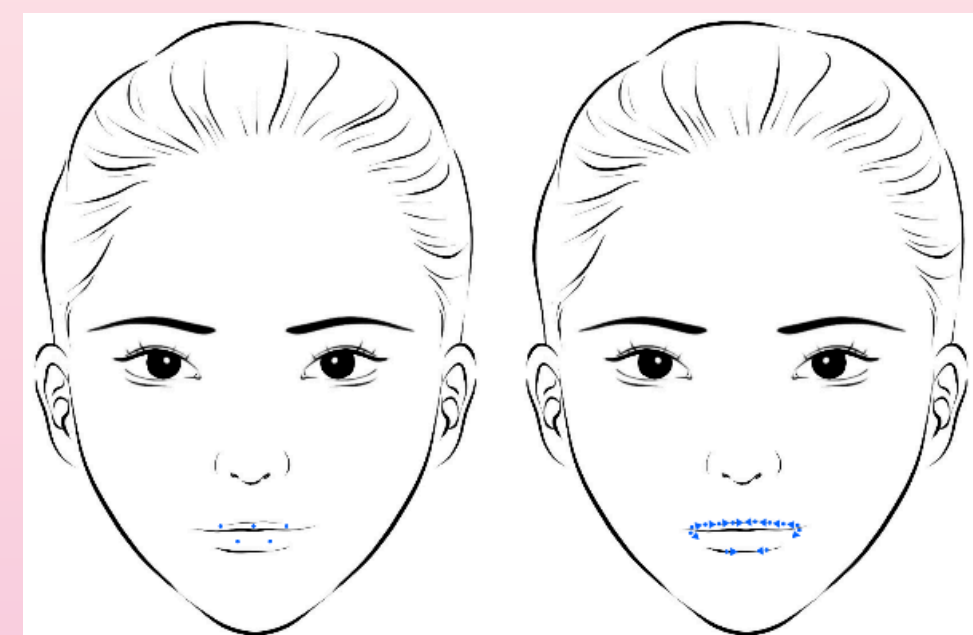


Example of procedure

Pretarsal roll - Classic

Cannula: 27G cannula

Needle: 30G Needle

Injection amount: 0.4cc-0.8cc

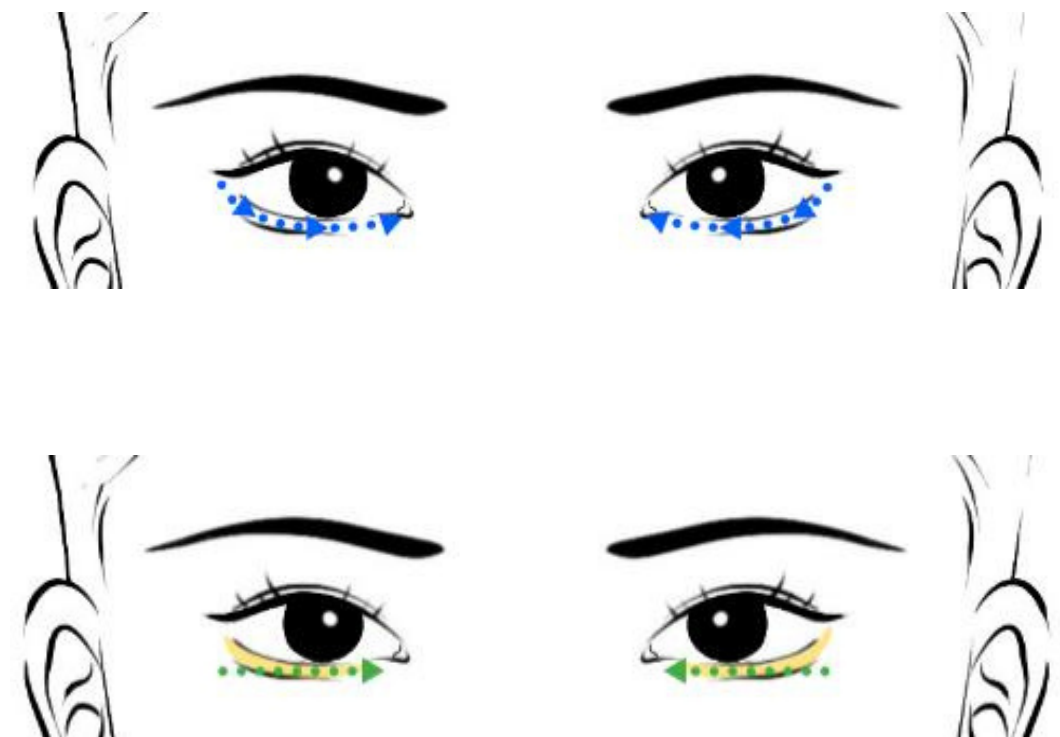


Chin - Volume

Cannula: 25 or 27G cannula

Needle: 25 or 27G Needle

Injection amount: 2cc (Min 1cc- Max 4cc)

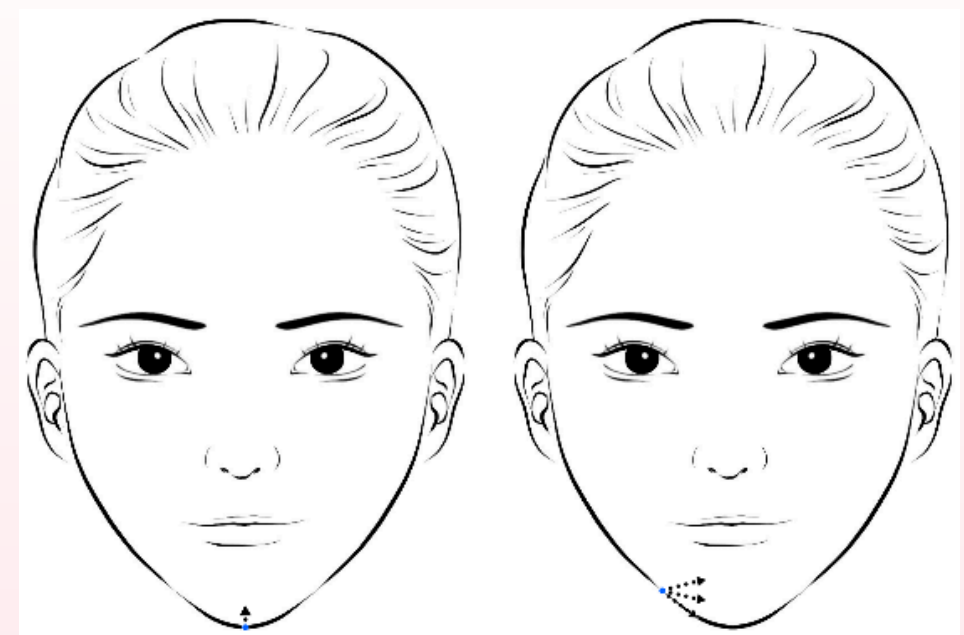


Nose - Volume

Cannula: 23 or 25G cannula

Needle: 25 or 27G Needle

Injection amount: 1cc (Min 0.5cc- Max 1.3cc)

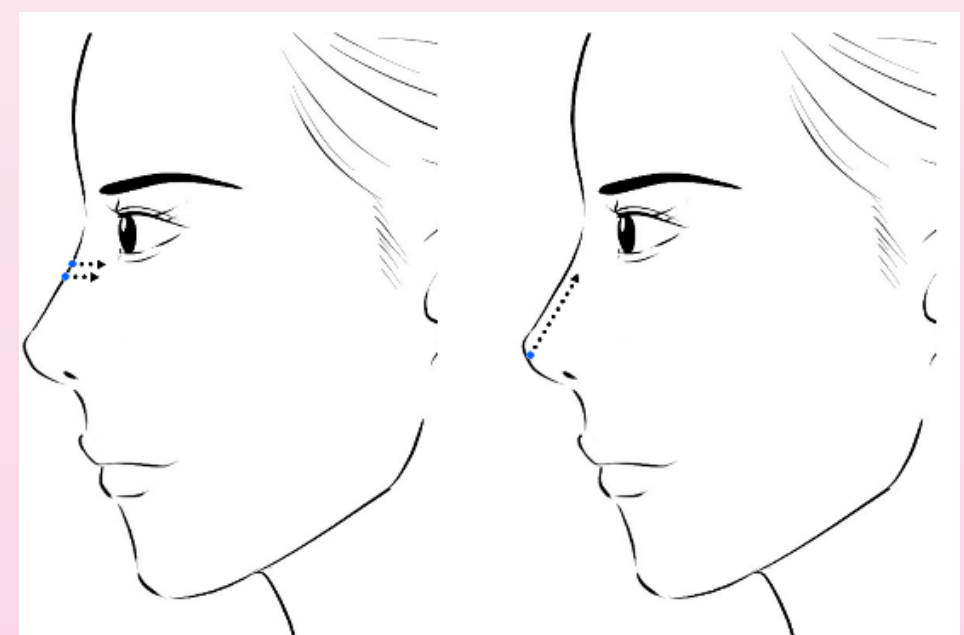

Certification

CE Certified

- CE marking confirms that the product complies with European Union safety, health, and regulatory requirements for distribution within the EU market.



ISO Certified

- Manufactured in facilities certified under ISO standards, ensuring consistent quality management and internationally recognized production practices.



PURAVELLE^{CE}₂₂₆₅

Cross Linked Hyaluronic Acid

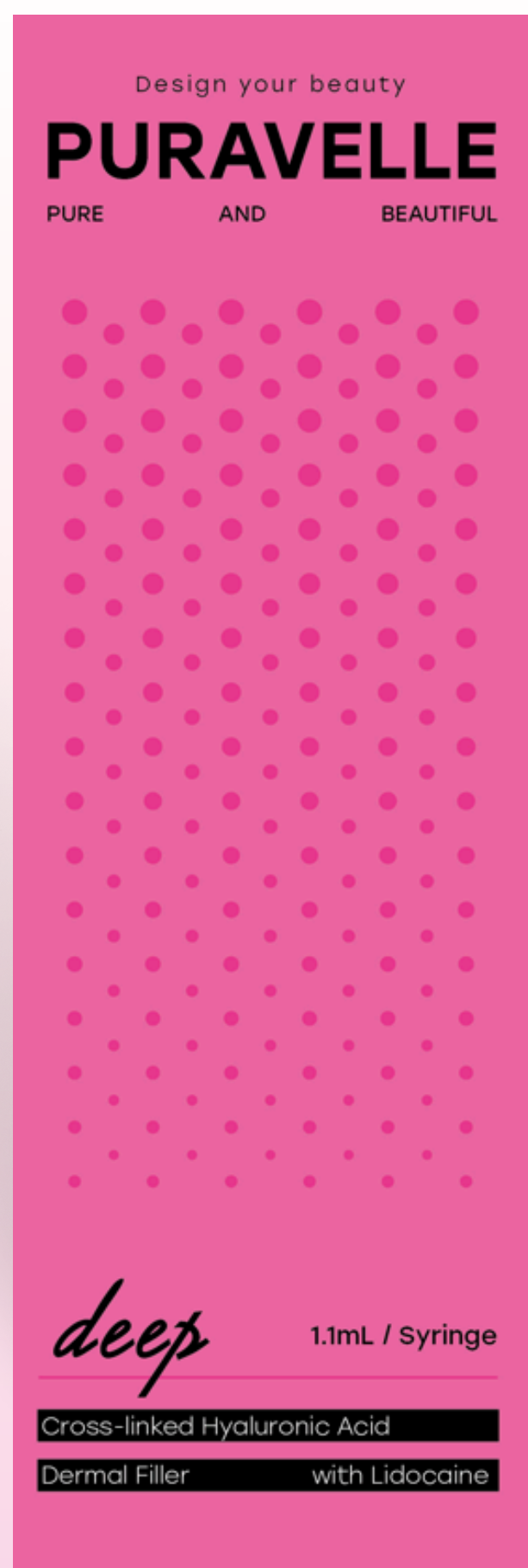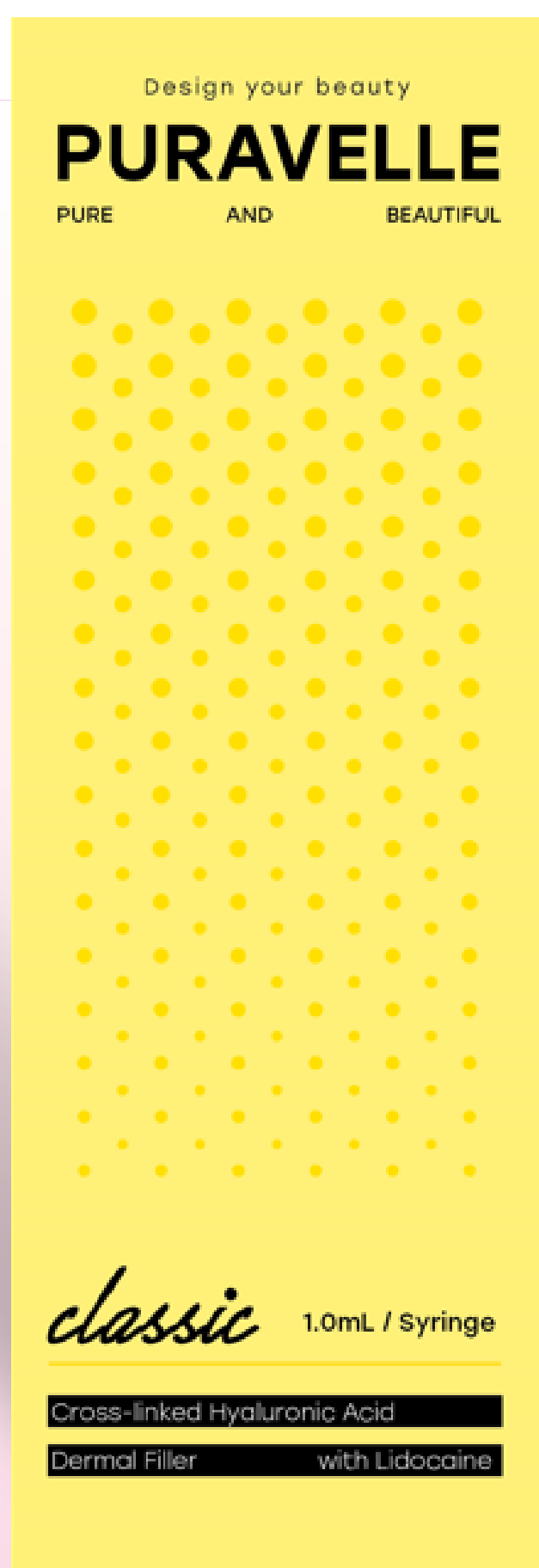

WEBSITE



DAESUNG MEDI Co., Ltd.

#211, 2nd floor, SKn Tehno Park, Mega building,
124, Sagimakgol-ro, Jungwon-gu, Seongnam-si,
Gyeonggi-do, Republic of Korea

✉ dsmedi2020@naver.com

☎ +82 10-3395-5944