



Under Eye Area Rejuvenation with PN



LUNA EYES

KHICARE

SKIN REGENERATION

Under Eye Area Rejuvenation with PN

Luna Eyes contains Salmon DNA fragments that make up the polynucleotide (PN). It's beneficial for skin due to its regenerative and hydrating properties. PN promotes skin-cell growth and repair, enhances skin elasticity, reduces wrinkles, and provides deep hydration, leading to healthier, more youthful-looking skin.

Best Choice for *Shining You*




Specification of LUNA EYES



Validated Safety

Enhanced safety by using highly biocompatible tissue regeneration material PN and HA.

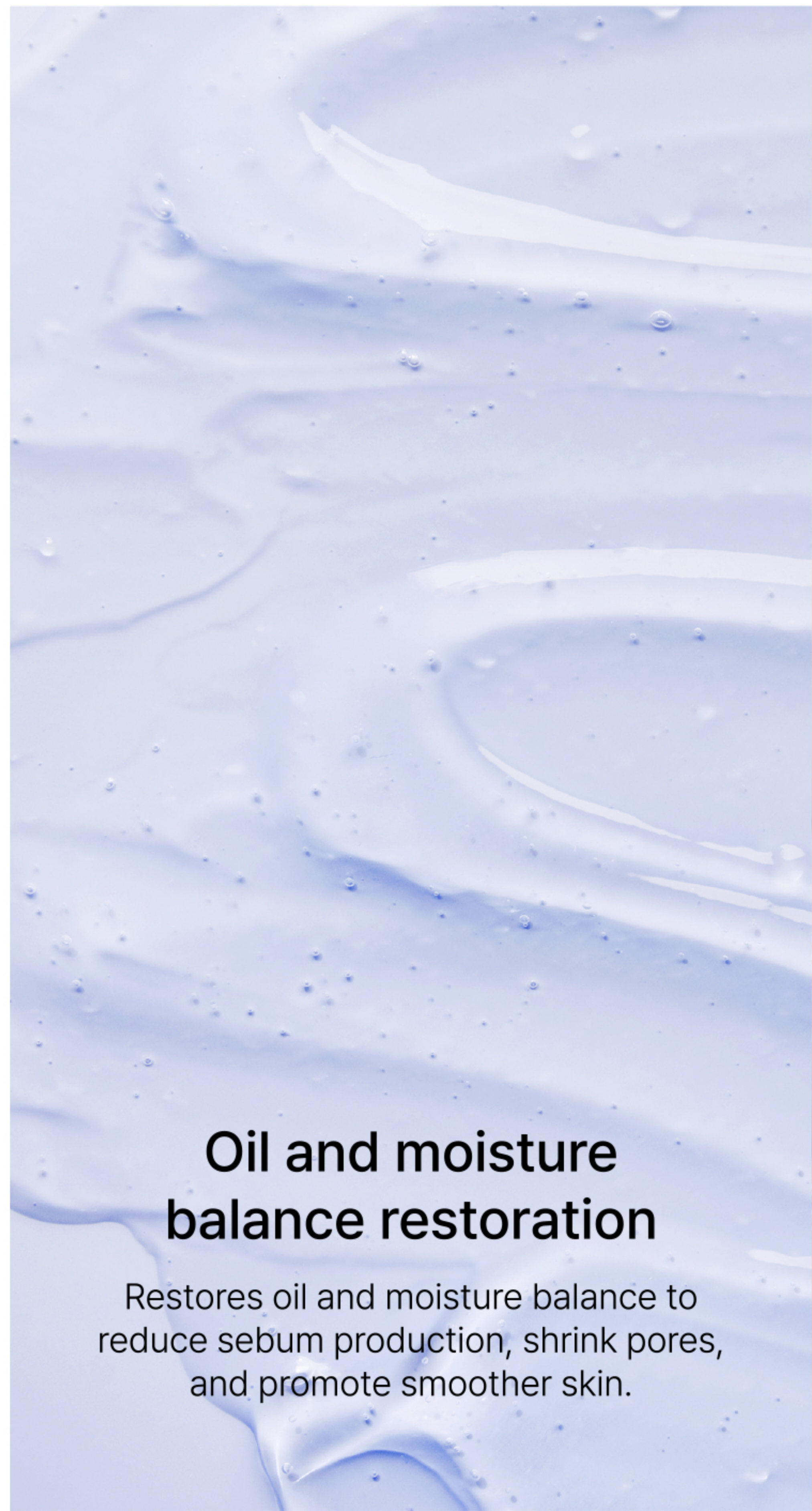

Oil and moisture balance restoration

Restores oil and moisture balance to reduce sebum production, shrink pores, and promote smoother skin.



Elasticity Improvement

Increase the density and thickness of the dermis to improve elasticity.



Skin Regeneration

Boosts self-sufficiency by activating the skin's unique regeneration capabilities.



Say Goodbye to Dark Circles

"LUNA EYES" contains anti-aging PN ingredient to reduce wrinkles, remove dark circles, and gives skin a younger and healthier look. (PN 1%)
Dry and dull skin? Do you need to improve your skin hydration and reduce wrinkles?
The best choice for a shining you is "LUNA EYES".

How it Works

Luna Eyes is very effective at improving wrinkles and dark circles. Notable results are achievable with a single treatment, yet continuous application will further enhance the radiance of your eye area.

Reduce Wrinkles



Remove Dark Circles



session
1st

interval
20~30days

session
2nd

interval
20~30days

session
3rd

Case 1.



[Before]



[Immediately after the procedure]



[After 3 days]

Case 2.



[Before]



[Immediately after the procedure]



[After 3 days]

Visible results can be seen even after the 1st treatment. For best results, 3-4 sessions at 20-30 days intervals are recommended.

Procedure


Intensive treatment provides your skin with brightness and a radiant glow.



Under eye treatment

1. Use 33g 4mm needle or Multi needle / Meso needle
2. Point 6~10 spots under one eye
3. Inject the same amount (1mL for one eye / 2mL for one session total)
4. Up to 3 treatments every 20 - 30 days

* You can inject more amount on the dark circle areas or wrinkled areas.
Product for professional use only.

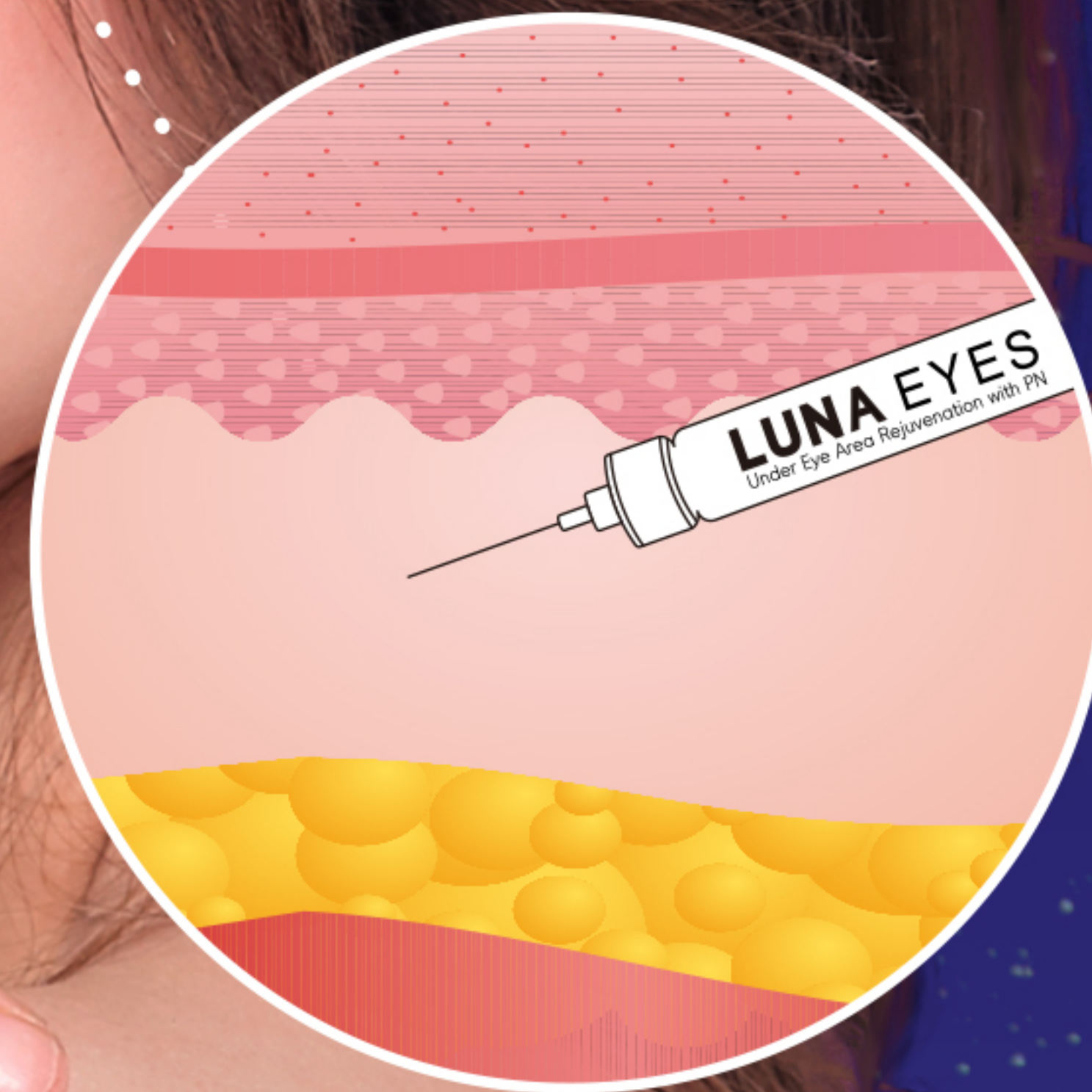

Product Information



Composition	PN (Polynucleotide)
Intended Use	Temporary improvement of fine wrinkles, skin elasticity, skin structure
Viscosity	Medium
Elasticity	Soft
Treatment Area	Facial, Eye Area
Injection Depth	Mid-deep dermis (Recommend)
Needle	33g 4mm
Packing Unit	1 syringe
Volume	2mL
Volume of contents	PN (10mg/mL)
Duration	4~6 months
Storage	Room Temperature
Origin	South Korea
The Period of Validity	3 Years



LUNA EYES

Under Eye Area Rejuvenation with PN

Company	KBH CARE
Items	Dermal fillers, botulinum toxins and others
Email	kbhcare@naver.com
Address	#228, 68, Digital-ro 9-gil, Geumcheon-gu, Seoul, Korea

